


Atos Medical Inc. Product Catalog



2015



ATOS
MEDICAL

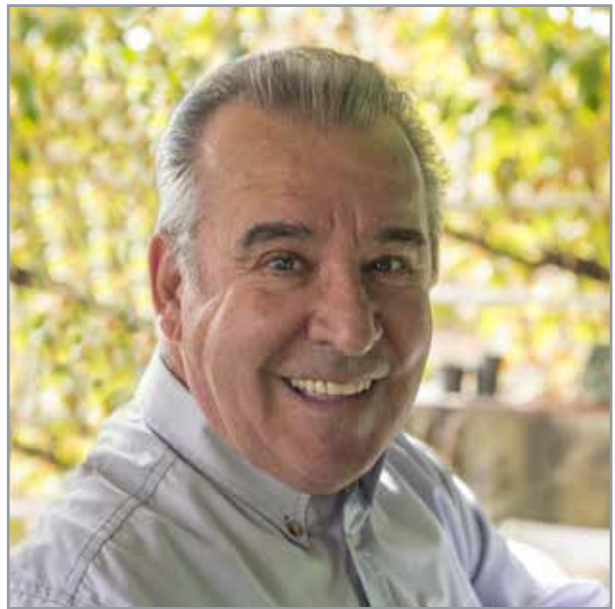

Contents

Provox Voice Prostheses	4
Specialty Voice Prostheses	7
Voice Prosthesis Accessories for Professionals	8
Voice Prosthesis Accessories for Patients	10
Provox HMEs and Speaking Valves	11
Accessories for HME Attachments	15
Provox Attachments	17
Accessories for Tracheostoma Support	22
Laryngectomy Pulmonary Kit (LPK)	25
CareTips	26
Anatomy Illustrations	27
Electrolarynges	28
TheraBite® Jaw Motion Rehabilitation System™	30
Selection of References	32


The Provox Essential Three: Voice Prosthesis, HME and Attachment

The solution for enhanced voice restoration and improved pulmonary health

The Provox System has been developed with voice-restoration and lung-rehabilitation in mind. The system offers three core product lines – the Provox voice prostheses, Provox HMEs and Provox attachments. We call it a system since the combination of products from all three product lines provides an even more powerful solution than the products as stand-alone devices. As in life, together we are stronger. Finding the right combination of Provox voice prosthesis, HME and attachment is key in giving each user a good voice and pulmonary health. Therefore, the Provox System has a wide range of products to assist people in finding the solution that works for them.

Provox Voice Prostheses

The Provox System offers a complete range of voice prostheses and accessories to provide users with good voice quality, low speaking effort, comfort, and easy device maintenance.

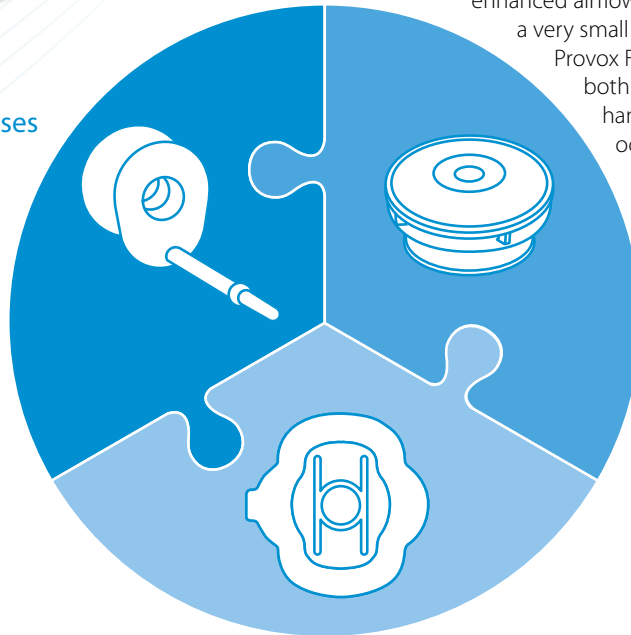

Provox HMEs and Speaking Valves

Provox XtraFlow and XtraMoist provide enhanced airflow and humidification in a very small and discreet design.

Provox FreeHands FlexiVoice offers both the possibility to speak hands-free and with manual occlusion.



Provox Attachments

The Provox System offers a complete range of attachment options to find the optimal solution. Atos Medical is a pioneer in this field, and we continuously strive to develop new patient-friendly products, all based on evidence and experience.



Provox Voice Prostheses

The Provox System offers a complete range of standard voice prostheses and accessories to provide users with good voice-quality, low speaking effort, comfort, and easy device-maintenance. In addition, several specialty voice prostheses and accessories are available for solving problems, such as early leakage through, and around, the voice prosthesis.



Provox® Vega™ Puncture Set

A disposable surgical set for primary and secondary placement of the Provox Vega voice prosthesis

The Provox Vega Puncture Set allows voice prosthesis insertion immediately after creating the tracheoesophageal (TE) puncture. Insertion is made in three steps - Protect, Puncture and Place.

Protect - The transparent pharynx protector (intended for primary puncture only) protects the pharyngeal walls.

Puncture - The puncture needle creates the puncture, and is used to introduce the GuideWire through the puncture site.

Place - The TE puncture is gradually dilated when the dilator, attached to the Provox GuideWire, is pulled through the same. The voice prosthesis, thereafter, slides gently into position.

In a multicenter clinical study, 100% of the voice prostheses could be placed using the Provox Vega Puncture Set; 89% of the procedures required no additional instruments.

- All-in-one surgical set
- Creates a round puncture, leaving the surrounding tissue intact
- Comes pre-loaded with a Provox Vega voice prosthesis
- Is disposable
- Enables a rapid procedure; minimum preparation and assembly are needed
- Is sterile



Provox Vega Puncture Set	Size	REF
Provox Vega Puncture Set 17Fr	8 mm	8140US
Provox Vega Puncture Set 17Fr	10 mm	8141US
Provox Vega Puncture Set 17Fr	12.5 mm	8142US
Provox Vega Puncture Set 17Fr	15 mm	8143US

Provox Vega Puncture Set	Size	REF
Provox Vega Puncture Set 20Fr	8 mm	8144US
Provox Vega Puncture Set 20Fr	10 mm	8145US
Provox Vega Puncture Set 20Fr	12.5 mm	8146US

Provox Vega Puncture Set	Size	REF
Provox Vega Puncture Set 22.5Fr	8 mm	8147US
Provox Vega Puncture Set 22.5Fr	10 mm	8148US
Provox Vega Puncture Set 22.5Fr	12.5 mm	8149US

Includes: 1 Pharynx Protector (made of transparent thermoplastic), 1 Puncture Needle (made of surgical stainless steel), 1 Guide Wire (made of pre-colored fluoroplastic), 1 Puncture Dilator with 1 preloaded Provox Vega voice prosthesis, 1 Provox Brush, 1 Provox Vega Plug and 1 Emergency Card.

For Professional Use Only

Provox® Vega™

A voice prosthesis designed for good voice quality, low speaking-effort, easy device-maintenance and comfort

Consistency, device lifetime, voice-quality and ease of voicing are important factors when choosing the right voice prosthesis. Clinical studies show that the Provox Vega has good device lifetime. A majority of users also said that the Provox Vega provides the best voice-quality, requires the lowest speaking-effort, and is the easiest, and most effective, to clean.

Most users also selected the Provox Vega as their preferred voice prosthesis over other voice prostheses.

Provox Vega speech samples are perceived by clinicians in blinded ratings:

- Less strained
- Easier to understand
- Being produced with less effort
- Having a better voice quality



Provox® Vega™ Voice Prosthesis with SmartInserter™

For fast and easy voice prosthesis replacement

The Provox Vega voice prosthesis comes pre-loaded inside the SmartInserter. No preparation or assembly is needed.

Features & Advantages

- Transparent, long tube for enhanced visibility
- Ribbed area on inserter pin for good grip
- Smooth folding of esophageal flange



Provox Vega	Rx	Size	REF
Provox Vega 17Fr	Rx	4 mm	8110
Provox Vega 17Fr	Rx	6 mm	8111
Provox Vega 17Fr	Rx	8 mm	8112
Provox Vega 17Fr	Rx	10 mm	8113
Provox Vega 17Fr	Rx	12.5 mm	8114
Provox Vega 17Fr	Rx	15 mm	8115

Provox Vega	Rx	Size	REF
Provox Vega 20Fr	Rx	4 mm	8120
Provox Vega 20Fr	Rx	6 mm	8121
Provox Vega 20Fr	Rx	8 mm	8122
Provox Vega 20Fr	Rx	10 mm	8123
Provox Vega 20Fr	Rx	12.5 mm	8124
Provox Vega 20Fr	Rx	15 mm	8125

Provox Vega	Rx	Size	REF
Provox Vega 22.5Fr	Rx	4 mm	8130
Provox Vega 22.5Fr	Rx	6 mm	8131
Provox Vega 22.5Fr	Rx	8 mm	8132
Provox Vega 22.5Fr	Rx	10 mm	8133
Provox Vega 22.5Fr	Rx	12.5 mm	8134
Provox Vega 22.5Fr	Rx	15 mm	8135

Sterile, for single use. Includes: 1 Provox Vega voice prosthesis pre-loaded in a single-use SmartInserter (sterile) and 1 Provox Brush (non-sterile)

Provox®2

The first voice prosthesis for anterograde insertion

Provox2 was introduced in 1997, and is the predecessor of the Provox Vega. The Provox2 can be inserted immediately after TE puncture (primary or secondary) using the Provox GuideWire. Replacement is carried out in an anterograde manner, using the single-use insertion tool.



Features & Advantages

- Low airflow resistance
- Easy maintenance
- Can be used for anterograde placement

Provox2	Rx	Size	REF
Provox2	Rx	4.5 mm	7216
Provox2	Rx	6 mm	7217
Provox2	Rx	8 mm	7218
Provox2	Rx	10 mm	7219

Provox2	Rx	Size	REF
Provox2	Rx	12.5 mm	7221
Provox2	Rx	15 mm	7224

Sterile, for single-use. Includes: 1 Provox2 Voice Prosthesis, 1 single-use insertion tool, 1 Provox Brush.

Provox® NiD™ Voice Rehabilitation System

The Provox NiD is a 'patient changable' voice prosthesis, meaning that the patient can manage cleaning and replacement of the device themselves.

Features & Advantages

- Patients may insert, remove and clean the device themselves
- Sturdy, durable materials and construction.
- The Provox NiD is attached to a safety medallion for additional safety
- Low airflow resistance
- In-situ cleaning possible with dedicated cleaning tools
- Available in 17Fr and 20Fr diameters, along with the most popular lengths



Provox NiD	Rx	Size	REF
Provox NiD 17	Rx	6 mm	7101
Provox NiD 17	Rx	8 mm	7102
Provox NiD 17	Rx	10 mm	7103
Provox NiD 17	Rx	12 mm	7104
Provox NiD 17	Rx	14 mm	7105
Provox NiD 17	Rx	18 mm	7106

Provox NiD	Rx	Size	REF
Provox NiD 20	Rx	6 mm	7111
Provox NiD 20	Rx	8 mm	7112
Provox NiD 20	Rx	10 mm	7113
Provox NiD 20	Rx	12 mm	7114
Provox NiD 20	Rx	14 mm	7115
Provox NiD 20	Rx	18 mm	7116

Inserters of corresponding size included. Non-sterile, ready for single patient use.

Specialty Voice Prostheses

Provox® Vega™ XtraSeal

Combining the benefits of the Provox Vega with a solution to manage leakage around

The Provox Vega XtraSeal is designed for users who experience leakage around the voice prosthesis due to enlarged TE punctures. It is a Provox Vega with an additional, enlarged esophageal flange.

Features & Advantages

- A voice prosthesis with a solution to manage leakage around
- The enlarged esophageal flange is thin and flexible, which allows it to conform to the tissue around the TE puncture
- The user can continue to speak, eat and drink while the leakage around is managed
- The valve-seat is radio opaque, which makes the voice prosthesis visible on xray



Provox Vega XtraSeal	Rx	Size	REF	Provox Vega XtraSeal	Rx	Size	REF	Provox Vega XtraSeal	Rx	Size	REF
Provox Vega XtraSeal 17Fr	Rx	4 mm	7770-01	Provox Vega XtraSeal 20Fr	Rx	4 mm	7776-01	Provox Vega XtraSeal 22.5Fr	Rx	4 mm	7782-01
Provox Vega XtraSeal 17Fr	Rx	6 mm	7771-01	Provox Vega XtraSeal 20Fr	Rx	6 mm	7777-01	Provox Vega XtraSeal 22.5Fr	Rx	6 mm	7783-01
Provox Vega XtraSeal 17Fr	Rx	8 mm	7772-01	Provox Vega XtraSeal 20Fr	Rx	8 mm	7778-01	Provox Vega XtraSeal 22.5Fr	Rx	8 mm	7784-01
Provox Vega XtraSeal 17Fr	Rx	10 mm	7773-01	Provox Vega XtraSeal 20Fr	Rx	10 mm	7779-01	Provox Vega XtraSeal 22.5Fr	Rx	10 mm	7785-01
Provox Vega XtraSeal 17Fr	Rx	12.5 mm	7774-01	Provox Vega XtraSeal 20Fr	Rx	12.5 mm	7780-01	Provox Vega XtraSeal 22.5Fr	Rx	12.5 mm	7786-01
Provox Vega XtraSeal 17Fr	Rx	15 mm	7775-01	Provox Vega XtraSeal 20Fr	Rx	15 mm	7781-01	Provox Vega XtraSeal 22.5Fr	Rx	15 mm	7787-01

Includes: 1 Provox Vega XtraSeal voice prosthesis pre-loaded in a single-use SmartInserter (sterile) and 1 Provox Brush (non-sterile).

Provox® ActiValve®

The voice prosthesis with extended device lifetime

The Provox ActiValve is designed for users who experience short prosthesis-lifetime (less than 4 – 8 weeks) caused by early leakage through the prosthesis or negative pressure in the pharynx/esophagus during swallowing or inhalation. The valve and valve-seat of the Provox ActiValve are closed with magnets, which prevents inadvertent opening.

Features & Advantages

- Reduces the need for frequent replacements in most patients who experience short device-lifetime
- Available in three strengths - Light, Strong and XtraStrong
- In a clinical study, the Provox ActiValve had a 16-fold device-life compared to standard voice prostheses



Provox ActiValve	Rx	Size	REF	Provox ActiValve	Rx	Size	REF	Provox ActiValve	Rx	Size	REF
Provox ActiValve Light	Rx	4.5 mm	7150	Provox ActiValve Strong	Rx	4.5 mm	7160	Provox ActiValve XtraStrong	Rx	4.5 mm	7165
Provox ActiValve Light	Rx	6 mm	7151	Provox ActiValve Strong	Rx	6 mm	7161	Provox ActiValve XtraStrong	Rx	6 mm	7166
Provox ActiValve Light	Rx	8 mm	7152	Provox ActiValve Strong	Rx	8 mm	7162	Provox ActiValve XtraStrong	Rx	8 mm	7167
Provox ActiValve Light	Rx	10 mm	7153	Provox ActiValve Strong	Rx	10 mm	7163	Provox ActiValve XtraStrong	Rx	10 mm	7168
Provox ActiValve Light	Rx	12.5 mm	7154	Provox ActiValve Strong	Rx	12.5 mm	7164	Provox ActiValve XtraStrong	Rx	12.5 mm	7169

Non-sterile, for single use. Includes: 1 Provox ActiValve voice prosthesis, 1 single-use insertion tool, 2 Provox Brush, 1 Provox ActiValve Lubricant, 1 Provox Plug, 2 User Cards, 1 Emergency Card.

Provox ActiValve Lubricant (replacement) 7149

Voice Prosthesis Accessories

for Professionals

Provox® GuideWire

The Provox GuideWire enables retrograde insertion of all Provox indwelling prostheses. The tip contains a retainer slit that holds the safety strap of the prosthesis.

Provox GuideWire	Quantity	REF
Provox GuideWire	1	7215

Sterile, for single use.



Provox® XtraFlange™

A rapid solution to manage leakage around

The Provox XtraFlange is a silicone washer, intended to reduce leakage around the voice prosthesis. For severe cases, the Provox XtraFlange can be used in combination with the Provox Vega XtraSeal. The security medallion is cut off after replacement, when the proper placement has been confirmed.

Features & Advantages

- Easy, quick, cost-effective solution to a common problem
- Can be used in combination with all Provox indwelling voice prostheses
- White material for good visibility

Provox XtraFlange	Rx	Quantity	REF
Provox XtraFlange 22.5 Fr	Rx	1	7275
Provox XtraFlange 20 Fr	Rx	1	7276
Provox XtraFlange 17 Fr	Rx	1	7277

Sterile, for single use.



Provox® Measure

The Provox Measure is used to determine the length (corresponding to prosthesis size) of TE punctures. The correct prosthesis size (1-2 mm longer than the puncture) is important to minimize the risk of complications.

Provox Measure	Quantity	REF
Provox Measure	1	7270
Provox Measure Flanges	5	7271

Provox Measure: non sterile, reusable, sterilizable.
Provox Measure Flanges: non sterile, for single use.



Provox® Dilator

The Provox Dilator is used to upsize the diameter of the TE puncture or to temporarily block a TE puncture, irrespective of type and diameter of voice prosthesis used. The size can be increased from 15 Fr to 24 Fr in one device (steps: 18, 20, 22, 24 Fr).

Provox Dilator	Quantity	REF
Provox Dilator	1	7211
Provox Dilator 17	1	7122
Provox Dilator 20	1	7123

Non-sterile, reusable, sterilizable. Two smaller dilators are also available; for dilating 17Fr or 20Fr punctures.



Voice Prosthesis Accessories

for Professionals

Provox® Capsules

Clinical studies show that the Provox® Vega™ voice prosthesis has superior device lifetime. Majority of users feel it provides the best voice quality, requires the lowest speaking effort, and is the easiest and most effective to clean.

The Provox Capsule gives you alternative options for voice prosthesis insertion methods. This means that you can combine the Provox Voice Prosthesis with the insertion method of your choice. Provox Capsules are made of a water soluble material of plant origin.

The Provox Capsule is available in the following sizes:

- Xtra Small (16 Fr)
- Small (17 Fr)
- Medium (20 Fr)
- Large (22.5 Fr)

The capsule can be used for anterograde insertion of a standard voice prosthesis of corresponding diameter.



Provox Capsules	Quantity	REF
Provox Capsules 16 Fr	15 pcs	7794
Provox Capsules 17 Fr	15 pcs	7795
Provox Capsules 20 Fr	15 pcs	7796
Provox Capsules 22.5 Fr	15 pcs	7797

Sterile, for single use.



Voice Prosthesis Accessories

for Patients

Provox® Brush

For cleaning of the voice prosthesis

The Provox Brush cleans the lumen and valve of the Provox voice prosthesis. Safety wings on the brush shaft help to prevent accidental aspiration of the Provox Brush. The Provox Brush is available in two lengths.

Features & Advantages

- Easy and efficient cleaning
- Can be bent for individual adaptation
- Also used to insert the Provox Plug or Provox Vega Plug



Provox Brush	Quantity	Size	REF
Provox Brush	6 pcs	4,5-10 mm	7204
Provox Brush XL	6 pcs	12-18 mm	7225

Non-sterile, reusable. For single patient use. For voice prostheses in sizes 12.5 and 15 mm use Provox Brush XL.

Provox® Flush

For flushing after brushing

The Provox Flush is a cleaning device with several attractive features. The Provox Flush allows flushing air or drinking water through the lumen of the prosthesis, thereby helping to clean debris from the prosthesis. Laboratory studies show that flushing with air may help to prolong device life.

Features & Advantages

- Can be used with air or drinking water
- May be bent for individual fitting



Provox Flush	Quantity	REF
Provox Flush	1 pc	8109

Non-sterile, reusable. For single patient use.

Provox® Plug & Provox® Vega™ Plug

A first-aid tool to temporarily stop leakage through the prosthesis

The Provox Plug is inserted into the opening of the Provox voice prosthesis with the help of the non-bristled end of a Provox Brush. The medallion end can be taped to the skin. Speaking is not possible while the Provox Vega Plug is in place. The plug can be removed by pulling the safety strap.

Note: The Provox Vega Plug fits into Provox Vega prostheses; the Provox Plug is for Provox2, Provox (1) and Provox ActiValve.

Provox Plug & Provox Vega Plug	Quantity	REF
Provox Plug	1 pc	7205
Provox Vega Plug, 17 Fr	1 pc	8119
Provox Vega Plug, 20 Fr	1 pc	8129
Provox Vega Plug, 22.5 Fr	1 pc	8139


Non-sterile, reusable. For single patient use. Includes Provox Brush.



Provox HMEs and Speaking Valves

Provox Heat and Moisture Exchangers (HMEs) provide better respiratory health by heating and moisturizing the inhaled air after a laryngectomy. They are the most extensively clinically tested and proven HMEs for laryngectomees in the world.

Provox FreeHands FlexiVoice makes it possible for more laryngectomees to speak hands-free.

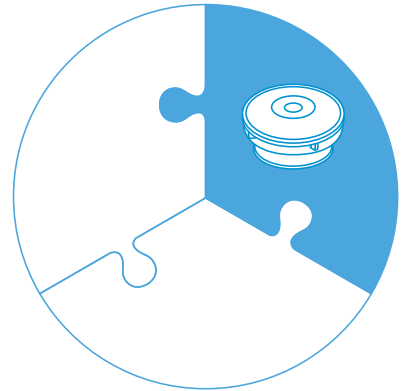

Provox® XtraHME™

Providing pulmonary rehabilitation in combination with speech

Provox XtraHME is available in two versions::

Provox XtraFlow - the HME choice for patients who need lower breathing resistance, for instance, during physical activity.

Provox XtraMoist - provides better humidification for patients who can tolerate the increased breathing-resistance.

- Easy occlusion of the stoma for voicing
- Reduced coughing and mucus production
- Can be used day and night up to 24 hours
- Low-profile design



Provox XtraHME	Quantity	REF
Provox XtraMoist HME	30	7290
Provox XtraFlow HME	30	7291

Provox® Micron HME™

An HME in combination with a highly-effective bacterial, viral and particulate matter filter

When breathing through a tracheostoma, particles that normally are filtered in the upper airways can enter the lower airways. Micron helps to filter small airborne particles, such as bacteria, viruses, dust and pollen.

Features & Advantages

- A highly effective bacterial and viral filter (≥99,8%)
- Reduced coughing and mucus production



Provox Micron HME	Rx	Quantity	REF
Provox Micron HME	Rx	5	7247
Provox Micron HME	Rx	30	7248



Provox HMEs and Speaking Valves

Provox® FreeHands FlexiVoice™

Gives you the freedom to speak hands-free

Provox FreeHands FlexiVoice offers both the possibility to speak hands-free and with manual occlusion. For hands-free speech, you can choose between three different strengths: light, medium, and strong. The choice depends on how much airflow and pressure is needed for speaking and breathing.

A clinical study has shown that more than 90% of users were able to speak hands-free with Provox FreeHands FlexiVoice; about half of the users used it every day.*

* Data on file.

- Choose between the light, medium or strong membrane devices
- Click-and-Go. Attach the speaking valve to the HME with a single click
- A hands-free speaking valve that also provides lung rehabilitation
- Choose between speaking hands-free or with manual occlusion



1 dot
Light membrane

2 dots
Medium membrane

3 dots
Strong membrane

Provox HMEs and Speaking Valves

Provox FreeHands FlexiVoice	Rx	REF
Provox FreeHands FlexiVoice Set Plus: 1 light, 1 medium, 1 strong, 2 arches, 1 Removal Aid, 30 HME Cassettes	Rx	7757
Provox FreeHands FlexiVoice Set: 1 light, 1 medium, 1 strong, 2 Arches, 1 Removal Aid	Rx	7760

Provox FreeHands FlexiVoice	Rx	REF
Provox FreeHands FlexiVoice Light: 1 light, 1 Arch, 1 Removal Aid	Rx	8161
Provox FreeHands FlexiVoice Medium: 1 medium, 1 Arch, 1 Removal Aid	Rx	8162
Provox FreeHands FlexiVoice Strong: 1 strong, 1 Arch, 1 Removal Aid	Rx	8163

Provox FreeHands FlexiVoice	Rx	REF
Provox FreeHands HME Moist: 30 pcs HME Cassettes	Rx	8220
Provox FreeHands HME Flow: 30 pcs HME Cassettes	Rx	8221



Provox HMEs and Speaking Valves

Accessories for HME Attachments

Remove™

Remove is used to eliminate any silicone glue residue after removing the adhesive. Remove can also be applied on top of the adhesive to make it easier to remove the adhesive. After use, the skin needs to be cleaned (water and non-oily soap or Provox Cleaning Towel) before using a new adhesive. Manufactured by Smith and Nephew Inc.



Remove	Quantity	REF
Remove	50 pcs	403100

Non-sterile, for single use.

Provox® Cleaning Towel

Provox Cleaning Towel cleans and removes oil from the skin. It is alcohol-based and non-perfumed.



Provox Cleaning Towel	Quantity	REF
Provox Cleaning Towel	200 pcs	7244

(20 resealable pouches of 10). Non sterile, for single use.

Skin-Prep™

Skin-Prep can improve the adhesion of Provox adhesives and protects the skin by forming a thin, protective barrier. Manufactured by Smith and Nephew Inc.



Skin-Prep	Quantity	REF
Skin-Prep	50 pcs	420400

Non-sterile, for single use.

Provox® Silicone Glue

Provox Silicone Glue is a liquid glue that can be used to improve the adhesion between the skin and the adhesive.

Provox Silicone Glue	Quantity	REF
Provox Silicone Glue 40 ml	1 pc	7720

Bottle with brush cap, non sterile. For multiple application.



Skin Tac™

Skin Tac improves adhesion of appliances and tape to the skin. It also works as a barrier between tape and skin to reduce irritation. Manufactured by Torbot Group Inc.



Skin Tac	Quantity	REF
Skin Tac Adhesive Barrier Wipes	50	MS407W

TacAway®

TacAway is ideal for removing glue and stubborn adhesive residue around the stoma when changing the Provox® Adhesives. Manufactured by Torbot Group Inc.

TacAway	Quantity	REF
TacAway	50	MS408W



Stoma Care®

Stoma Care Natural Aloe Based Skin Care wipes provide a soothing, moisturizing, non-greasy base for the skin around the stoma prior to the application of an adhesive and/or tape. Manufactured by Torbot Group Inc.

Stoma Care Wipes	Quantity	REF
Stoma Care Wipes	50	TT600



Provox® HME Cap

Provox HME Cap is used with the FreeHands HMEs. The Cap can be occluded manually for speech, and makes it possible for the FreeHands HMEs also to be used during sleep.

Provox HME Cap	Quantity	REF
Provox HME Cap matte titanium	1 pc	7730
Reusable, non sterile		



Provox® HME Cassette Adaptor

Provox HME Cassette Adaptor enables use of a Provox HME Cassette to any tracheostomy tube with a 15 mm ISO connector.

Provox HME Cassette Adaptor	Quantity	REF
Provox HME Cassette Adaptor	1 pc	7246
Non-sterile, for single patient use.		



Provox® ShowerAid

Provox ShowerAid is used with a Provox attachment while taking a shower.

Provox ShowerAid	Quantity	REF
Provox ShowerAid	1 pc	7260
Non-sterile, for single patient use. Includes: 1 Provox FlexiDerm Oval Adhesive (7254).		



Provox® BasePlate Adaptor


Provox BasePlate Adaptor makes it possible to attach a device, e.g. an HME, with an ISO 15 mm standard connector to a Provox attachment.

Provox BasePlate Adaptor	Quantity	REF
Provox BasePlate Adaptor	1 pc	7263
Non-sterile, for single patient use.		



Provox Attachments

Differences in anatomy, skin types and situations of use are important when finding a suitable attachment for the HME or FreeHands Speaking Valve. The Provox System offers a complete range of attachments.

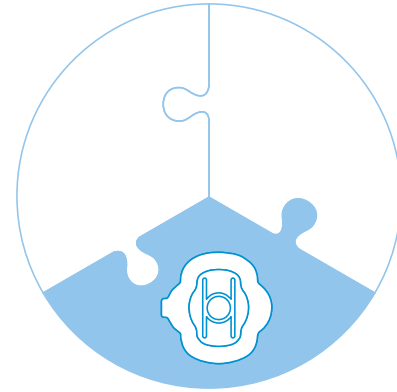

Provox® StabiliBase™

Especially suited for hands-free speaking with Provox FreeHands FlexiVoice

The innovative Provox StabiliBase design simplifies application, provides support during hands-free speech, and increases adhesive lifetime for a majority of users.

The Provox StabiliBase is also available in OptiDerm material - Provox StabiliBase OptiDerm - intended for users with sensitive skin.

Provox StabiliBase	Quantity	REF
StabiliBase	1	7299
StabiliBase	15	7289
StabiliBase OptiDerm	5	7328
StabiliBase OptiDerm	15	7318

Non-sterile, for single use.

- Especially suited for hands-free speaking
- Improved fitting in deep tracheostomas
- Strong adhesion to the skin



Provox® XtraBase®

Due to the special shape and reinforced center, Provox XtraBase is ideal for users with a deep tracheostoma.

The Provox XtraBase has a strong adhesion to the skin, and can be used for hands-free speaking.

Provox XtraBase	Quantity	REF
Provox XtraBase	20	7265

Non-sterile, for single use.



Provox® Adhesives

Our Oval and Round adhesives are mainly intended for users with a regular tracheostoma and skin. With an extra large surface contact area, the Plus design is intended for users who require a larger adhesive that provides more support.

The Provox FlexiDerm has a strong adhesion, but is yet soft and flexible. This makes Provox FlexiDerm suitable for people with a regular tracheostoma who require strong adhesion.

Provox Adhesive FlexiDerm	Quantity	REF
FlexiDerm Round	20	7253
FlexiDerm Oval	20	7254
FlexiDerm Plus	20	7331

Non-sterile, for single use.



FlexiDerm Round



FlexiDerm Oval



FlexiDerm Plus

The Provox OptiDerm is made of a skin-friendly hydrocolloid material, and is recommended for users with a temporary or chronic skin sensitivity. Provox OptiDerm is also especially suitable for the first days after surgery.

Provox Adhesive OptiDerm	Quantity	REF
OptiDerm Round	20	7255
OptiDerm Oval	20	7256
OptiDerm Plus	20	7332

Non-sterile, for single use.



OptiDerm Round



OptiDerm Oval



OptiDerm Plus

The Provox® Regular is a transparent adhesive with normal adhesion and easy to remove.

Provox Adhesive Regular	Quantity	REF
Regular Round	20	7251
Regular Oval	20	7252
Regular Plus	20	7330

Non-sterile, for single use.




Regular Round



Regular Oval



Regular Plus

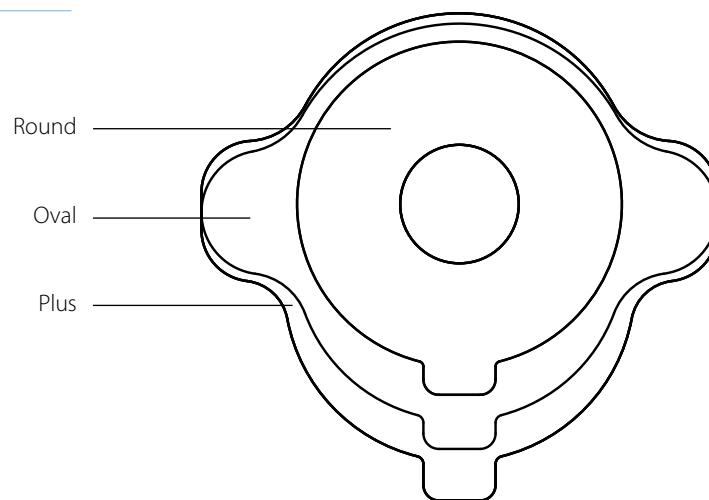

Provox® LaryTube™

The Provox LaryTube is a soft silicone laryngectomy tube that maintains the opening of the tracheostoma. It can be used to attach a Provox HME or a Provox FreeHands Speaking Valve. The Provox LaryTube is well suited for immediate postoperative Provox HME use, HME use during radiotherapy, and HME use in patients with sensitive skin. Provox LaryTube is available in Standard, Fenestrated and with Ring (to be secured with Provox adhesive).

Standard	Rx	ID	OD	L	REF
LaryTube 8/27	Rx	9.5	12.0	27	7601
LaryTube 8/36	Rx	9.5	12.0	36	7602
LaryTube 8/55	Rx	9.5	12.0	55	7603
LaryTube 9/27	Rx	10.5	13.5	27	7605
LaryTube 9/36	Rx	10.5	13.5	36	7606
LaryTube 9/55	Rx	10.5	13.5	55	7607

Standard	Rx	ID	OD	L	REF
LaryTube 10/27	Rx	12.0	15.0	27	7609
LaryTube 10/36	Rx	12.0	15.0	36	7610
LaryTube 10/55	Rx	12.0	15.0	55	7611
LaryTube 12/27	Rx	13.5	17.0	27	7613
LaryTube 12/36	Rx	13.5	17.0	36	7614
LaryTube 12/55	Rx	13.5	17.0	55	7615

includes: 5 XtraFlow Cassettes. Non-sterile, for single use.



Fenestrated	Rx	ID	OD	L	REF
LaryTube 8/36	Rx	9.5	12.0	36	7637
LaryTube 8/55	Rx	9.5	12.0	55	7638
LaryTube 9/36	Rx	10.5	13.5	36	7640
LaryTube 9/55	Rx	10.5	13.5	55	7641

Fenestrated	Rx	ID	OD	L	REF
LaryTube 10/36	Rx	12.0	15.0	36	7643
LaryTube 10/55	Rx	12.0	15.0	55	7644
LaryTube 12/36	Rx	13.5	17.0	36	7646
LaryTube 12/55	Rx	13.5	17.0	55	7647

includes: 5 XtraFlow Cassettes and 1 Provox Brush. Non-sterile, for single use.



With Ring	Rx	ID	OD	L	REF
LaryTube 8/36	Rx	9.5	12.0	36	7624
LaryTube 8/55	Rx	9.5	12.0	55	7625
LaryTube 9/36	Rx	10.5	13.5	36	7626
LaryTube 9/55	Rx	10.5	13.5	55	7627

With Ring	Rx	ID	OD	L	REF
LaryTube 10/36	Rx	12.0	15.0	36	7628
LaryTube 10/55	Rx	12.0	15.0	55	7629
LaryTube 12/36	Rx	13.5	17.0	36	7630
LaryTube 12/55	Rx	13.5	17.0	55	7631

includes: 5 XtraFlow Cassettes, 1 FlexiDerm Oval. Non-sterile, for single use.



Provox® LaryTube™ Sizer Kit

To allow for proper selection and sizing, the Provox LaryTube Sizer Kit contains sterilizable and reusable tubes for individual fitting. The products can be secured with Provox LaryClip and Provox TubeHolder.

Provox LaryTube Sizer Kit	Quantity	REF
Provox Larytube Sizer Kit	1	7648

Non-sterile, reusable, sterilizable (Sizer Tubes and inner box). Outer tray can be cleaned in instrument washer.

Provox StomaMeasure	Quantity	REF
StomaMeasure for LaryTube	1	7807

Non-sterile - education material



Provox® LaryButton™

The Provox LaryButton is a silicone stoma button that maintains the opening of the tracheostoma. It can be used to attach an HME, also in combination with Provox FreeHands Speaking Valve. The Provox LaryButton is well suited for patients with a circumferential skin fold around the tracheostoma (a so-called "lip") and patients with sensitive skin.



Provox LaryButton	Rx	OD	L	REF
Provox LaryButton 12/8	Rx	12	8	7671
Provox LaryButton 14/8	Rx	14	8	7672
Provox LaryButton 16/8	Rx	16	8	7673
Provox LaryButton 18/8	Rx	18	8	7674

Provox LaryButton	Rx	OD	L	REF
Provox LaryButton 12/18	Rx	12	18	7685
Provox LaryButton 14/18	Rx	14	18	7686
Provox LaryButton 16/18	Rx	16	18	7687
Provox LaryButton 18/18	Rx	18	18	7688

Non-sterile, for single patient use. Maximum device lifetime 6 months.

Provox® LaryButton™ Sizer Kit


To allow for proper selection and sizing, the Provox LaryButton Sizer Kit contains sterilizable and reusable tubes for individual fitting. The products can be secured with Provox LaryClip and Provox TubeHolder.

Provox LaryButton Sizer Kit	Quantity	REF
Provox LaryButton Sizer Kit	1	7690


Non-sterile, reusable, sterilizable (Sizer Buttons and inner box). Outer tray can be cleaned in instrument washer.



Provox Attachments

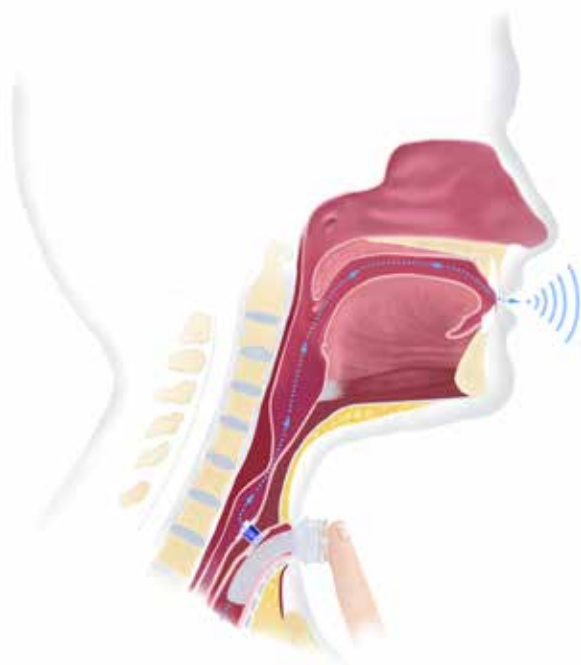

Speaking with the Provox voice prosthesis:
Manual occlusion of the Provox XtraHME attached with a Provox LaryTube.

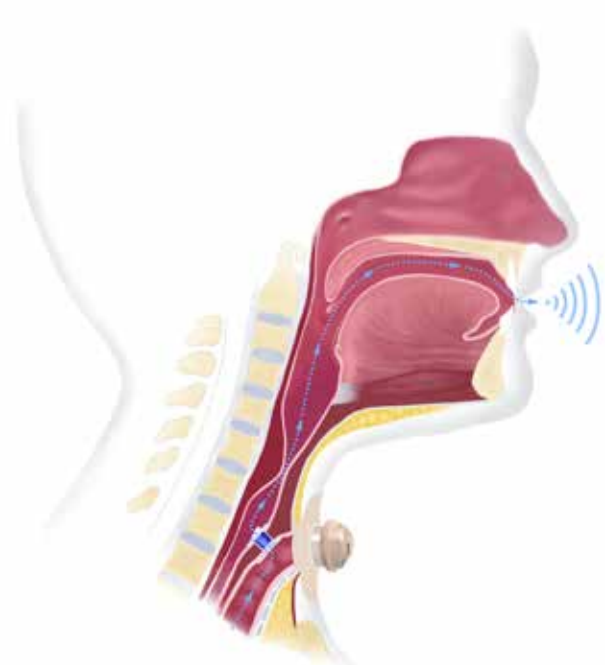

Speaking with the Provox voice prosthesis:
Handsfree speech using a Provox LaryButton and Provox FreeHands Speaking valve.

Barton-Mayo™ Tracheostoma Button

The Barton-Mayo Tracheostoma Button (Mayo Clinic) was developed to provide leakproof tracheostoma valve-retention for near-total laryngectomy and total laryngectomy patients. Manufactured by Medical Innovations International, Inc.



Features & Advantages

- Can be used with an HME (Heat and Moisture Exchanger)
- Enhances speech with “hands-free” stoma occlusion
- Another option for positive stoma sealing
- Reduces stoma contractions with a wide range of sizes available

The Barton-Mayo Buttons listed below may not have been licensed in accordance with Canadian law.

Order info	Size	Models	Diameter (mm)	Length (mm)	Rx	REF
Barton-Mayo Tracheostoma Button	9	Short	13.5	15	Rx	ENBMB091
		Regular		22		ENBMB092
		Long		30		ENBMB093
Barton-Mayo Tracheostoma Button	10	Short	15.0	15	Rx	ENBMB101
		Regular		22		ENBMB102
		Long		30		ENBMB103
Barton-Mayo Tracheostoma Button	12	Short	17.0	15	Rx	ENBMB121
		Regular		22		ENBMB122
		Long		30		ENBMB123
Barton-Mayo Tracheostoma Button	14	Short	19.0	15	Rx	ENBMB141
		Regular		22		ENBMB142
		Long		30		ENBMB143
Barton-Mayo Sizing Kit		Includes trial buttons for each available Barton-Mayo Button				ENBMBKIT

Conversion chart

Barton-Mayo™ Button				Provox LaryButton		
REF		Diameter	Length	REF	Diameter	Length
9	Short	13.5	15	7671	12	8
	Regular	13.5	22	7685	12	18
	Long	13.5	30			
10	Short	15.0	15	7672	14	8
	Regular	15.0	22	7686	14	18
	Long	15.0	30			
12	Short	17.0	15	7673	16	8
	Regular	17.0	22	7687	16	18
	Long	17.0	30			
14	Short	19.0	15	7674	18	8
	Regular	19.0	22	7688	18	18
	Long	19.0	30			

Accessories for Tracheostoma Support

Provox® Fenestration Punch

The Provox Fenestration Punch is used to make small fenestrations in the Provox LaryTube.



Provox Fenestration Punch	Quantity	REF
Provox Fenestration Punch	1 pc	7654

Non-sterile, reusable, sterilizable.

Provox® TubeBrush

The Provox TubeBrush is used to clean the Provox LaryTube and Provox LaryButton.



Provox TubeBrush	Quantity	Size	REF
Provox TubeBrush	6 pc	8 mm	7660
Provox TubeBrush	6 pc	12 mm	7661

Non-sterile, reusable. For single patient use.

Kapi-Gel® Tracheostoma Spacer

Kapi Gel® Tracheostoma Spacer is a 43 mm diameter, 3 or 5 mm thick circular, tacky, laminated silicone gel spacer. It is available in two center hole openings to accommodate different diameter tubes and Buttons (8 mm opening fits tubes or buttons with <14 mm OD; 12 mm opening fits tubes or buttons with >12 mm OD). Kapi-Gel spacers are intended to help create a better seal around tracheostoma support devices, and are suitable for uneven stoma anatomy. Kapi-Gel can improve tracheostoma device fitting, act as a cushion behind tubes and buttons during digital occlusion, and may help reduce air leakage. Manufactured by Kapitex Healthcare Ltd.



Kapi-Gel Tracheostoma Spacer	Quantity	REF
8 mm ID, 3mm Thick	10 pc	LATNG2008
12mm ID, 3mm Thick	10 pc	LATNG2014
8mm ID, 5mm Thick	10 pc	LATNG4008
12mm ID, 5mm Thick	10 pc	LATNG4012

Non-sterile, for single patient use.

Provox® TubeHolder

The Provox TubeHolder has been developed exclusively for use with Provox LaryTube and Provox LaryButton. It is suitable for users with sensitive skin around the tracheostoma. The integrated clip connectors allow for optimal fit to the wings of the Provox LaryTube and LaryButton.



Provox TubeHolder	Quantity	REF
Provox TubeHolder	1 pc	7668

Non-sterile, reusable. For single patient use.

Provox® LaryClip

The Provox LaryClip is an alternative to Provox TubeHolder. It is a two-piece system that helps to optimize the air-tight attachment of Provox LaryButton and Provox LaryTube to the tracheostoma.



Provox LaryClip	REF
1 set: 8 pcs LaryClip, 40 pcs LaryClip Base	7669
Non-sterile, for single patient use.	



Stoma Covers

Features & Advantages

- 100% Polyester 4-ply fabric
 - Discreet coverage
 - Seven versatile colors
 - Easily Adjustable Neck Strap from 13 1/4 to 18 inches
 - Machine Washable
 - Dimension: Approx. 9 1/2 inches wide and 8 1/4 inches long
- Manufactured by Fountain Head Marketing, Inc.



Stoma Covers		
Item Number	Color	Material
RSC WH	WHITE	4-PLY Polyester
RSC BE	BEIGE	4-PLY Polyester
RSC NA	NAVY	4-PLY Polyester
RSC GR	GRAY	4-PLY Polyester
RSC LB	TURQUOISE	4-PLY Polyester
RSC BU	BURGUNDY	4-PLY Polyester
RSC BL	BLACK	4-PLY Polyester

Mock Turtle Neck Covers

Features & Advantages

- Mock Turtle Neck Style for discreet coverage
 - Comfortable 80/20 Poly/Cotton Blend resists shrinkage and fading
 - Machine Washable
 - Three Available Colors
 - Adjustable neck collar from 12 to 21 inches
 - Dimensions: Approx. 10 inches wide and 8 inches long
- Manufactured by Fountain Head Marketing, Inc.



Mock Turtle Neck Covers		
Item Number	Color	Material
RTNSC BL	BLACK	Cotton/Poly Blend
RTNSC NA	NAVY	Cotton/Poly Blend
RTNSC WH	WHITE	Cotton/Poly Blend

Foam Stoma Covers

Features & Advantages

- Lightweight Medical-Grade Foam
 - Available in Beige
 - One Adhesive Strip
- Manufactured by Fountain Head Marketing, Inc.



Foam Stoma Covers				
Item Number	Color	Adhesives	Size	Pcs
RFFFL02	BEIGE	1 strip	65 x 65 mm	30 pcs

Laryngectomy Pulmonary Kit (LPK)

Immediate post-operative pulmonary care enhances patient outcomes

Clinical studies have shown that most laryngectomized patients who use a Provox Heat and Moisture Exchanger (HME) 24 hours a day, even while sleeping, experience reductions in:

- Mucus Production
- Coughing or Forced Expectorations

Immediate postoperative use of the HME also lowers costs as compared to the use of an external humidifier.

Provox Laryngectomy Pulmonary Kits provide convenient, all-in-one customizable packages supporting pulmonary care as well as communication needs. Standard and custom packaging options are available.

Standard Pulmonary Kit:

- Provox® XtraFlow HME™ (30 pcs)
- Provox® LaryTube™ (standard)
- Provox® Tubeholder
- Provox® TubeBrush (6 pcs)
- Provox® ShowerAid
- Care Tips educational materials for the Laryngectomee



Custom Options Available:

- Provox® Vega™ Voice Prosthesis
- Provox® Guidewire
- Provox® LaryTube™ (standard, fenestrated or with ring)
- Provox® OptiDerm™ Adhesives
- Artificial Larynges: Liberty™, SolaTone™, TruTone™
- Stoma Shower Guard
- Provox® Vega™ Puncture Set




To discuss the option that best fits your needs contact your Atos Medical Customer Representative at 1.800.217.0025.

CareTips

The CareTips Patient Information Kit is a toolkit for healthcare professionals to use with their patients prior to, and after their laryngectomies. This kit is given to patients to assist them as they move forward with their new lives as laryngectomees.

This kit presents the opportunity for the patient to register with Atos Medical and continuously receive FREE:

- Atos Medical Service Benefits
- Product samples
- Product discounts
- Laryngectomee and caregiver educational support

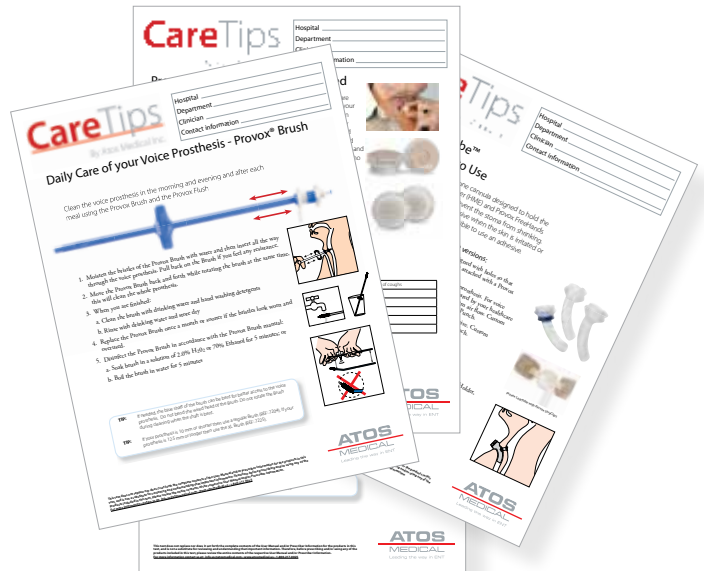

Order information	REF
CareTips Patient Information Kit	ML067

“Life as a Laryngectomee” Brochure

This introduction to post-laryngectomy rehabilitation explains the rationale behind products. Contact Atos Medical Customer Service to receive this brochure in a wide spectrum of languages.


Order information	REF
Life as a Laryngectomee	7962US

Instructional CD's and DVD's

Order information	REF
A Practical Guide to Post-Laryngectomy Rehabilitation – including the Provox System (CD-ROM)	ISBN 90-75575-05-X
Laryngectomy Pulmonary Kit DVD	MC1302
Provox HME DVD	7969-N
Picture CD-ROM	7819
Olfaction CD	ISBN 90-75575-07-6
Rescue Breathing for Emergency Medical Personnel and First Responders	ML075
TheraBite Instructional DVD	7840-N



Laryngectomy Illustrations

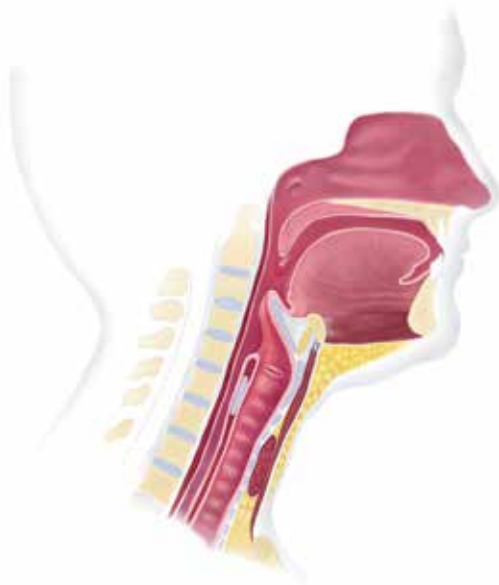


Anatomy before laryngectomy



Speaking with the Provox voice prosthesis:
Manual occlusion of the Provox XtraHME
attached with a Provox LaryTube.



Anatomy after laryngectomy

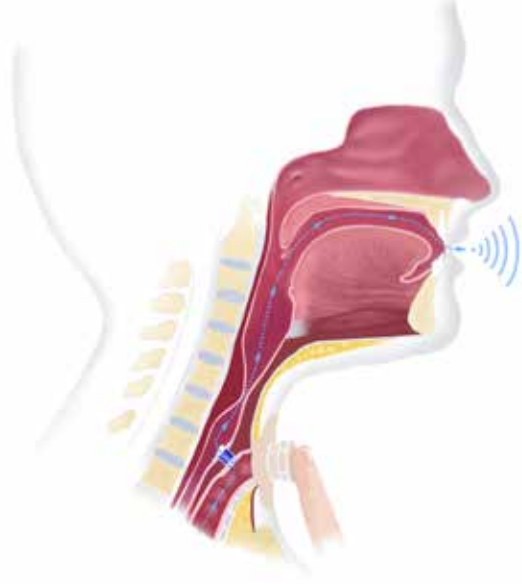

Speaking with the Provox voice prosthesis:
Manual occlusion of the Provox XtraHME
attached with an adhesive.

Electrolarynges

Atos Medical is proud to offer the full line of electrolarynges. Our customers have the freedom to choose a primary or back-up electronic speech aid with the features, pricing and warranties that can only be found through Atos Medical.



THE BEST!*

TruTone™ Electric Speech Aid

The only instrument available with natural intonation.

Features & Advantages

- Single Pressure-Sensitive Button for Natural Expression
- Pitch adjustment allows for masculine or feminine voice
- Light Weight, Small Size and Extremely Durable
- 9v Rechargeable and Standard Battery Operation
- Atos Medical Exclusive Full 5 Year Drop and Soak Warranty

Manufactured by Griffin Laboratories.

TruTone Electric Speech Aid	REF
TruTone Electric Speech Aid with 9v Battery Charger, Oral Adapter, Straws, Lanyard and Batteries	K101
TruTone Electric Speech Aid Featuring USB Charging	K120



BETTER*

SolaTone™ Electric Speech Aid

Features a wide frequency range.

Features & Advantages

- Light weight, small size and durable
- 9v Rechargeable and Standard Battery Operation
- Exclusive Full 3 Year Drop and Soak Warranty

Manufactured by Griffin Laboratories.

SolaTone Electric Speech Aid	REF
SolaTone Electric Speech Aid with 9v Battery Charger, Oral Adapter, Straws, Lanyard and Batteries	K276
SolaTone Electric Speech Aid Featuring USB Charging	K220



GOOD*

Liberty™ Electric Speech Aid

Atos Medical's exclusive value-priced speech aid.

Features & Advantages

- External Volume and Pitch
- AA battery Operation
- 3 Year Manufacturer Warranty

Manufactured by Griffin Laboratories.

Liberty Electric Speech Aid	REF
Liberty Electric Speech Aid with 9v Battery Charger, Oral Adapter, Straws, Lanyard and Batteries	K306

*Comparisons are within the line of Griffin Products.

Servox® Inton Speech Aid

Features & Advantages

- Titanium Sleeve
- Adjustable Pitch
- External Volume Wheel
- Base and Accentuation Buttons
- Comes with Setting Screwdriver
- 2 Year Warranty

Manufactured by Servona GmbH.

Servox Inton Speech Aid	REF
Servox Inton Speech Aid with LA2.40 Charger, NiMH Battery, Oral Adapter, Straws, Hip Bag and Carrying Cord	15529



Servox® Digital Speech Aid

Features & Advantages

- Programmable Base and Accentuation Settings
- Titanium Sleeve
- Adjustable Pitch
- External Volume Rocker Switch
- 2 Year Warranty

Manufactured by Servona GmbH.

Servox Digital Speech Aid	REF
Servox Digital Speech Aid with LA2.40 Charger, NiMH Battery, Oral Adapter, Straws, Hip Bag and Carrying Cord	17140



TheraBite® Jaw Motion Rehabilitation System™

The TheraBite® Jaw Motion Rehabilitation System™ is a portable system which utilizes repetitive passive motion and stretching to restore mobility and flexibility of the jaw musculature, associated joints and connective tissues. It is a clinically-proven, effective system that helps to increase mouth opening and decrease pain.



Exercise Log

The daily and monthly Exercise Log enables the user and his/her treating clinician to log the daily and monthly progress.



Bite Pads

The self-adhesive Bite Pads spread the force across the teeth and can be trimmed for customized protection. The Bite Pads come in adult, pediatric and edentulous (toothless) versions.



Range of Motion Scales

The disposable Range of Motion Scales allows the patient and his/her clinician to monitor the progress of the treatment.



Order information	Quantity	REF
TheraBite® Jaw Motion Rehabilitation System™, Adult	1 pc	TH001
TheraBite® Jaw Motion Rehabilitation System™, Pediatric	1 pc	TH002
The TheraBite® Jaw Motion Rehabilitation System™ contents:		
• The Jaw Mobilizer	1 pc	
• Bite Pads	4 pcs	
• Hand-Aid	1 pc	
• Range of Motion Scale	30 pcs	
• Carrying Bag	1 pc	
• Instructions for Use	1 pc	
• Patient Progress Log	1 pc	
• TheraBite Instructional DVD	1 pc	
TheraBite® ActiveBand	1 pc	8260
TheraBite® Range of Motion Scale	150 pcs	SC001
TheraBite® Bite Pads, Regular	4 pcs	PA001
TheraBite® Bite Pads, Edentulous	4 pcs	PA002
TheraBite® Bite Pads, Pediatric	4 pcs	PA003
TheraBite Instructional DVD		
This 14-minute film provides useful information on trismus, and demonstrates the functions and benefits of the TheraBite system. This DVD is divided into several chapters, and features testimonials from a former patient and two clinicians. Available in English only.	1 pc	7840-N



Carrying Bag

The durable nylon Carrying Bag keeps the TheraBite system together.



Hand-Aid

The Hand-Aid assists the patient in maintaining constant opening during stretching or strengthening programs.

Range Setting Arm

The position of the Range Setting Arm can be adjusted to 25-45 mm, and is used to determine maximum opening appropriate to each stage of rehabilitation.

Fine Adjustment Knob

The Fine Adjustment Knob can be used to adjust opening distance, if needed.

Selection of References

Provox® Voice Prostheses

- Lewin, J. S., Portwood, M. A., Wang, Y., and Hutcheson, K. A. Clinical application of the Provox NiD voice prosthesis: a longitudinal study. *Laryngoscope*, 124(7), 1585-1591. 2014.
- Kress P, Schafer P, Schwerdtfeger FP, Rosler S. Are modern voice prostheses better? A lifetime comparison of 749 voice prostheses. *Eur Arch Otorhinolaryngol*. 2014 Jan;271(1):133-40
- Hilgers FJ, Lorenz KJ, Maier H et al. Development and (pre-) clinical assessment of a novel surgical tool for primary and secondary tracheoesophageal puncture with immediate voice prosthesis insertion, the Provox Vega Puncture Set. *Eur Arch Otorhinolaryngol* 2013 Jan;270(1):255-62.
- Hancock K, Ward E, Lawson N, van As-Brooks CJ. A prospective, randomized comparative study of patient perceptions and preferences of two types of indwelling voice prostheses. *Int J Lang Commun Disord*. 2012 May-Jun;47(3):300-9.
- Ward EC, Hancock K, Lawson N, van As-Brooks CJ. Perceptual characteristics of speech production using the new indwelling Provox Vega voice prosthesis: A randomized controlled cross-over trial. *Head & Neck*. 2011 Jan;33(1):13-9.
- Hilgers FJ, Ackerstaff AH, Jacobi I, Balm AJ, Tan B, Van den Brekel MW. Prospective clinical phase II study of two new indwelling voice prostheses (Provox Vega 22.5 and 20 Fr) and a novel anterograde insertion device (Provox SmartInsersetter). *Laryngoscope*. 2010 Jun;120(6):1135-43.
- Hilgers FJ, Ackerstaff AH, van RM et al. Clinical phase I/ feasibility study of the next generation indwelling Provox voice prosthesis (Provox Vega). *Acta Otolaryngologica* 2010 April; 130(4):511-9.
- Hancock K, Houghton B, Van As-Brooks CJ, Coman W: First clinical experience with a new nonindwelling voice prosthesis (Provox NiD) for voice rehabilitation after total laryngectomy. *Acta Otolaryngologica* 2005 Sep;125(9):981-90.
- Makitie AA, Niemensivu R, Juvas A, Aaltonen LM, Back L, Lehtonen H: Postlaryngectomy voice restoration using a voice prosthesis: a single institution's ten-year experience. *Annals of Otolaryngology, Rhinology and Laryngology* 2003 Dec;112(12):1007-10.
- Hilgers FJ, Ackerstaff AH, Balm AJ, Van den Brekel MW, Bing Tan I, Persson JO: A new problemsolving indwelling voice prosthesis, eliminating the need for frequent Candida- and "underpressure"-related replacements: Provox ActiValve. *Acta Otolaryngologica* 2003 Oct;123(8):972-9.
- Free RH, Van der Mei HC, Elving GJ, Van Weissenbruch R, Albers FW, Busscher HJ: Influence of the Provox Flush, blowing and imitated coughing on voice prosthetic biofilms in vitro. *Acta Otolaryngologica* 2003 May;123(4):547-51.
- Jessica Soolsma, MSc; Michiel W. van den Brekel, MD, PhD; Annemieke H. Ackerstaff, PhD; Alfons J. Balm, MD, PhD; Bing Tan, MD, PhD; Frans J. Hilgers, MD, PhD Long-Term Results of Provox ActiValve, Solving the Problem of Frequent Candida- and "Underpressure"-Related Voice Prosthesis Replacements *The Laryngoscope* 2007 *The American Laryngological, Rhinological and Otolaryngical Society, Inc.*
- Op de Coul BMR, Hilgers FJM, Balm AJM, Tan IB, van den Hoogen FJA, van Tinteren H: A decade of post-laryngectomy vocal rehabilitation in 318 patients: A single institution's experience with consistent application of Provox® indwelling voice prostheses. *Archives of Otolaryngology- Head & Neck Surgery* 2000 Nov;126(11):1320-8.
- Balle VH, Rindsø L, Thomsen JC. Primary speech restoration at laryngectomy by insertion of voice prosthesis – 10 years experience. *Acta Otolaryngologica* 2000; Suppl 543:244-245.
- Ackerstaff AH, Hilgers FJ, Meeuwis CA, van der Velden LA, van den Hoogen FJ, Marres HA, Vreeburg GC, Manni JJ: Multi-institutional assessment of the Provox®2 voice prosthesis. *Archives of Otolaryngology - Head & Neck Surgery* 1999 Feb;125(2):167-73.

- Hilgers FJM, Ackerstaff AH, Balm AJM, Tan IB, Aaronson NK, Persson JO. Development and clinical evaluation of a second-generation voice prosthesis (Provox®2) designed for anterograde and retrograde insertion. *Acta Otolaryngologica* 1997 Nov;117(6):889-96.
- Hilgers FJM, Schouwenburg PF. A new low-resistance, self-retaining prosthesis (Provox®) for voice rehabilitation after total laryngectomy. *Laryngoscope* 1990 Nov;100(11):1202-7.

Provox® HME and Provox FreeHands HME

- van den Boer C, Muller SH, Vincent AD, Züchner K, van den Brekel MW, Hilgers FJ. Ex vivo water exchange performance and short-term clinical feasibility assessment of newly developed heat and moisture exchangers for pulmonary rehabilitation after total laryngectomy. *Eur Arch Otorhinolaryngol*. 2014 Feb;271(2):359-66.
- van den Boer C, Vas Nunes JH, Muller SH, van der Noort V, van den Brekel MW, Hilgers FJ. Water Uptake Performance of Hygroscopic Heat and Moisture Exchangers after 24-Hour Tracheostoma Application. *Otolaryngol Head Neck Surg*. 2014 Mar 28;150(6):999-1004.
- van den Boer C, Muller SH, Vincent AD, van den Brekel MW, Hilgers FJ. Ex vivo assessment and validation of water exchange performance of 23 heat and moisture exchangers for laryngectomized patients. *Respir Care*. 2014 Aug;59(8):1161-71.
- Herranz J, Espiño MA, Morado CO. Pulmonary rehabilitation after total laryngectomy: a randomized cross-over clinical trial comparing two different heat and moisture exchangers (HMEs). *Eur Arch Otorhinolaryngol*. 2013 Sep;270(9):2479-84.
- Brook I, Bogaardt H, van As-Brooks C. Long-term use of heat and moisture exchangers among laryngectomees: medical, social, and psychological patterns. *Ann Otol Rhinol Laryngol*. 2013 Jun;122(6):358-63.
- Hilgers FJ, Dirven R, Wouters Y, Jacobi I, Marres HA, van den Brekel MW. A multicenter, prospective, clinical trial evaluating a novel adhesive baseplate (Provox StabiliSeal) for peristomal attachment of postlaryngectomy pulmonary and voice rehabilitation devices. *Laryngoscope*. 2012 Nov;122(11):2447-53.
- Scheenstra RJ, Muller SH, Hilgers FJ. Endotracheal temperature and humidity in laryngectomized patients in a warm and dry environment and the effect of a heat and moisture exchanger. *Head Neck*. 2011 Sep;33(9):1285-93.
- Scheenstra RJ, Muller SH, Vincent A, Ackerstaff AH, Jacobi I, Hilgers FJ. A new heat and moisture exchanger for laryngectomized patients: endotracheal temperature and humidity. *Respir Care*. 2011 May;56(5):604-11.
- Scheenstra RJ, Muller SH, Vincent A, Hilgers FJ. Heat and moisture exchange capacity of the upper respiratory tract and the effect of tracheotomy breathing on endotracheal climate. *Head Neck*. 2011 Jan;33(1):117-24.
- Scheenstra RJ, Muller SH, Vincent A, Sinaasappel M, Hilgers FJ. Influence of breathing resistance of heat and moisture exchangers on tracheal climate and breathing pattern in laryngectomized individuals. *Head & Neck* 2010 Aug;32(8):1069-78.
- Scheenstra RJ, Muller SH, Vincent A, Ackerstaff AH, Jacobi I, Hilgers FJM. Short term endotracheal climate changes and clinical effects of a heat and moisture exchanger with an integrated electrostatic virus and bacterial filter developed for laryngectomized individuals. *Acta Otolaryngologica* . 2010 Jun;130(6):739-46
- Bien S, Okla S, van As-Brooks CJ, Ackerstaff AH. The effect of a Heat and Moisture Exchanger (Provox HME) on pulmonary protection after total laryngectomy: a randomized controlled study. *European Archives of Otorhinolaryngology* 2010 March;267(3):429-35.
- Lorenz KJ, Huverstuhl K, Maier H. [Finger-free speech with the Provox FreeHands HME Automatic Tracheostoma Valve system. Clinical long-term experience]. *HNO* 2009 Nov;57(11):1090-8.
- Zuur JK, Muller SH, Vincent A, Sinaasappel M, de Jongh FH, Hilgers FJ. The influence of a heat and moisture exchanger on tracheal climate in a cold environment. *Medical & Biological Engineering & Computing* 2009 Sep;31(7):852-7.
- Lorenz KJ, Maier H. [Pulmonary rehabilitation after total laryngectomy using a heat and moisture exchanger (HME)]. *Laryngorhinootologie* 2009 Aug;88(8):513-22.
- Scheenstra RJ, Muller SH, Vincent A, Sinaasappel M, Zuur JK, Hilgers FJ. Endotracheal temperature and humidity measurements in laryngectomized patients: intra- and inter-patient variability. *Medical & Biological Engineering & Computing* 2009 Jul;47(7):773-82.
- Zuur JK, Muller SH, Vincent A, Sinaasappel M, de Jongh FH, Hilgers FJ. Assessment of tracheal temperature and humidity in laryngectomized individuals and the influence of a heat and moisture exchanger on tracheal climate. *Head & Neck* 2008 Aug;30(8):1072-1082.
- Hilgers FJ, Soolsma J, Ackerstaff AH, Balm FJ, Tan IB, van den Brekel MW. A thin tracheal silicone washer to solve periprosthetic leakage in laryngectomies: direct results and long-term clinical effects. *Laryngoscope*. 2008 Apr;118(4):640-5.
- Zuur JK, Muller SH, Sinaasappel M, Hart GA, Van Zandwijk N, Hilgers FJ. Influence of heat and moisture exchanger respiratory load on transcutaneous oxygenation in laryngectomized individuals: a randomized cross-over study. *Head & neck* 2007 Dec;29(12):1102-1110.
- Lorenz KJ, Ackerstaff AH, Hilgers FJM, Maier H. Hands-free speech after surgical voice rehabilitation with a Provox® voice prosthesis: experience with the Provox® Freehands™ HME tracheostoma valve system. *European Archives of Otorhinolaryngology* 2007 Feb; 264(2):151-157.
- Hilgers, F. J. & Ackerstaff, A. H. Development and evaluation of a novel tracheostoma button and fixation system (Provox LaryButton and LaryClip adhesive) to facilitate hands-free tracheoesophageal speech. *Acta Otolaryngologica* 2006 Dec;126(11):1218-1224.
- Hamade, R., Hewlett, N., & Scanlon, E. A quantitative and qualitative evaluation of an automatic occlusion device for tracheoesophageal speech: the Provox FreeHands HME. *Clin Linguist Phon*. 2006 Apr-May;20(2-3):187-93.
- Zuur JK, Muller SH, de Jongh FH, van Zandwijk N, Hilgers FJ: The physiological rationale of heat and moisture exchangers in post-laryngectomy pulmonary rehabilitation: a review. *European Archives of Otorhinolaryngology* 2006 Jan;263(1):1-8.
- Tervonen H, Back L, Juvas A, Rasanen P, Makitie AA, Sintonen H, Roine RP, Vilkanen E, Aaltonen LM: Automatic speaking valve in speech rehabilitation for laryngectomized patients. *European Archives of Otorhinolaryngology* 2005 Oct;262(10):816-20.
- Op de Coul BM, Ackerstaff AH, van As-Brooks CJ, van den Hoogen FJ, Meeuwis CA, Manni JJ, Hilgers FJ. Compliance, quality of life and quantitative voice quality aspects of hands-free speech. *Acta Otolaryngologica* 2005 Jun;125(6):629-37.
- Ackerstaff AH, Fuller D, Irvin M, Maccracken E, Gaziano J, Stachowiak L: Multicenter study assessing effects of heat and moisture exchanger use on respiratory symptoms and voice quality in laryngectomized individuals. *Otolaryngology Head & Neck Surgery* 2003 Dec;129(6):705-12

- Hilgers FJ, Ackerstaff AH, Van As CJ, Balm AJ, Van den Brekel MW, Tan IB. Development and clinical assessment of a heat and moisture exchanger with a multi-magnet automatic tracheostoma valve (Provox FreeHands HME) for vocal and pulmonary rehabilitation after total laryngectomy. *Acta Otolaryngologica* 2003 Jan;123(1):91-9
- Herranz Gonzalez-Botas J, Suarez T, Garcia Carreira B, Martinez Moran A. Experience with the HME-Provox® Stomafilter in laryngectomized patients. *Acta Otorrinolaringologica Espana* 2001 Apr;52(3):221-5 [Spanish]
- van As CJ, Hilgers FJM, Koopmans-van Beinum FJ, Ackerstaff AH. The influence of stoma occlusion on aspects of alaryngeal speech. *Acta Otolaryngologica* 1998 Sep;118(5):732-8.
- Ackerstaff AH, Hilgers FJM, Balm AJM, Tan IB. Long term compliance of laryngectomized patients with a specialized pulmonary rehabilitation device, Provox® [HME] Stomafilter. *Laryngoscope* 1998 Feb;108(2):257-60.
- Hilgers FJM, Ackerstaff AH, Balm AJM, Gregor RT. A new heat and moisture exchanger with speech valve (Provox® [HME] Stomafilter). *Clinical Otolaryngology* 1996 Oct;21(5):414-8.

TheraBite Jaw® Motion Rehabilitation System™

- Retèl VP(1), van der Molen L, Steuten LM, van den Brekel MW, Hilgers FJ. A cost-effectiveness analysis of using TheraBite in a preventive exercise program for patients with advanced head and neck cancer treated with concomitant chemo-radiotherapy. *Eur Arch Otorhinolaryngol*. 2015 Feb 11. [Epub ahead of print]
- Kraaijenga S, van der Molen L, van Tinteren H, Hilgers F, Smeele L. Treatment of myogenic temporomandibular disorder: a prospective randomized clinical trial, comparing a mechanical stretching device (TheraBite®) with standard physical therapy exercise. *Cranio*. 2014 Jul;32(3):208-16. doi:
- Pauli N(1), Andréll P, Johansson M, Fagerberg-Mohlin B, Finizia C. Treating trismus: A prospective study on effect and compliance to jaw exercise therapy in head and neck cancer. *Head Neck*. 2014 Jul 2. doi: 10.1002/hed.23818. [Epub ahead of print]
- Kamstra JI, Roodenburg JL, Beurskens CH, Reintsema H, Dijkstra PU. TheraBite exercises to treat trismus secondary to head and neck cancer. *Support Care Cancer*. 2013 Apr;21(4):951-7 Epub 2014 Jan 24 .
- Ren WH(1), Ao HW, Lin Q, Xu ZG, Zhang B . Efficacy of mouth opening exercises in treating trismus after maxillectomy. *Chin Med J (Engl)*. 2013 Jul;126(14):2666-9.
- Carnaby-Mann G, Crary MA, Schmalzfuss I, Amdur R. "Pharyngocise": randomized controlled trial of preventative exercises to maintain muscle structure and swallowing function during head-and-neck chemoradiotherapy. *Int J Radiat Oncol Biol Phys* 2012;83(1):210-219.
- van der Molen L, vanRossum M, Smeele L, Rasch C, Hilgers F. A randomized controlled trial on preventive swallowing exercises in advanced head and neck cancer treated with chemoradiotherapy: 1-year functional outcomes. Preventive rehabilitation in patients treated with chemoreadiation for advanced head and neck cancer. Thesis ed. Amsterdam: Nederlands Kanker Instituut; 2011.
- van der Molen L(1), van Rossum MA, Burkhead LM, Smeele LE, Rasch CR, Hilgers FJ. A randomized preventive rehabilitation trial in advanced head and neck cancer patients treated with chemoradiotherapy: feasibility, compliance, and short-term effects. *Dysphagia*. 2011 Jun;26(2):155-70. doi: 10.1007/s00455-010-9288-y. Epub 2010 Jul 11.
- Dijkstra PU(1), Sterken MW, Pater R, Spijkervet FK, Roodenburg JL. Exercise therapy for trismus in head and neck cancer. *Oral Oncol*. 2007 Apr;43(4):389-94. Epub 2006 Sep 18.

- Cohen EG(1), Deschler DG, Walsh K, Hayden RE. Early use of a mechanical stretching device to improve mandibular mobility after composite resection: a pilot study. *Arch Phys Med Rehabil*. 2005 Jul;86(7):1416-9.
- Voisin, van Reck. Evaluation of the efficacy of a passive jaw motion device on patients with acute and chronic pain in the TMJ. Brussels 2005.
- Dijkstra PU(1), Kalk WW, Roodenburg JL. Trismus in head and neck oncology: a systematic review. *Oral Oncol*. 2004 Oct;40(9):879-89.
- Messing BP, Saunders JR, Dietrich-Burns K, Farrell SS, Pyrke-Fairchild J, Melick CF. Trismus management in the head and neck cancer patient undergoing organ preservation treatment. 6th International Conference on Head and Neck Cancer, Washington DC, USA August 07-11, 2004. Scientific Poster 01-97.
- Maloney GE(1), Mehta N, Forgione AG, Zawawi KH, Al-Badawi EA, Driscoll SE. Effect of a passive jaw motion device on pain and range of motion in TMD patients not responding to flat plane intraoral appliances. *Cranio*. 2002 Jan;20(1):55-66.
- Maloney GE, Mehta N, Forgione AG, Zazawi KH, Al-Badawi EA, Driscoll SE. Effects of a passive jaw motion device on pain and range of motion in TMD patients not responding to flat plane intraoral appliances. *The Journal of CranioMandibular Practice*, 2002; 20:55-66.
- Stack BC, Gregory E, Gjerde G, Hanssen JJ, Leivseth G. Refractive patients: TMJ splint therapy combined with open joint surgery: 60 patients; 117 joints. *Funct Orthod* 2002;18(4):24-36.
- Buchbinder D, Currivan RB, Kaplan AJ, Urken ML. Mobilization regimens for the prevention of jaw hypomobility in the radiated patient: a comparison of three techniques. *J Oral Maxillofac Surg* 1993;51(8):863-867.

The above references represent a selection. For more information, contact Atos Medical AB or your local representative.



Prescription Devices

Caution:

United States federal law restricts devices identified herein with the symbol "Rx" to sale, distribution and use by or on order of a physician or licensed health care provider. The availability of these devices without prescription outside the United States may vary from country to country.

Insurance coverage, payment, co-payments, deductibles and other restrictions vary depending on the patient's policy and medical need.

User Manual and Prescriber Information

This catalog does not replace nor does it set forth the complete contents of the User Manual and/or Prescriber Information for the products in this catalog, and is not a substitute for reviewing and understanding that important information. Therefore, before prescribing and/or using any of the products included in this catalog, please review the entire contents of the respective User Manual and/or Prescriber Information.

User Manuals and Prescriber Informations are available online at www.atosmedical.com. In the United States, you may also obtain copies by calling 1-800-217-0025, sending a Fax to 1-414-227-9033 or an e-mail to info.us@atosmedical.com, or writing to Atos Medical Inc., 11390 W. Theodore Trecker Way, West Allis, WI 53214-1135, U S A. Information in French is available online at www.atosmedical.com.

Limited Warranty:

Atos Medical makes no warranty, either expressed or implied, as to the lifetime of the product delivered to the purchaser hereunder, which may vary with individual use and biological conditions. Further, Atos Medical makes no warranty of merchantability or fitness of the product for any particular purpose.

Please Note:

The persons depicted in this document have agreed to be photographed for purposes of illustration only.



Celebrating 25 Years of Provox



Since the introduction of the first Provox voice prosthesis 25 years ago, Atos Medical has become the world leader in Laryngectomy Care. Our commitment and dedicated specialization in this field has allowed us to gain a deep understanding of the needs of our customers. This understanding drives our innovation process. We take pride in continuously bringing innovative solutions to our users.

Over the past 25 years we have served the needs of over 70,000 laryngectomized patients in more than 70 countries. Our comprehensive Laryngectomy Care portfolio includes a complete range of Provox voice and pulmonary rehabilitation devices. This broad range of high quality Provox products allows for a personalized combination of solutions for each user.

Atos Medical understands that being the leader in Laryngectomy Care comprises more than product development. Supporting clinical research and educating professionals and users are integral parts of our business. Our products are featured in over 150 peer-reviewed publications. We support and conduct several hundred conferences and educational programs every year.

We are proud of our history and accomplishments and we are excited to both continue learning from our users, and to serve them with the best products, services, and education in Laryngectomy Care.

Patents and Trademarks:

Provox®, Provox® ActiValve®, Provox® FreeHands HME®, Provox® XtraBase®, TrachPhone®, Provox® Stabilibase™ and Provox® XtraFlange™ are registered trademarks owned by Atos Medical AB, Sweden.
Provox® Vega™, Provox® SmartInserter™, Provox® NID™, Provox® XtraHME™, Provox® XtraMoist™ HME, Provox® XtraFlow™ HME, Provox® Micron HME™, Provox® FreeHands FlexiVoice™, Provox® HME Cap™, Provox® XtraFlange™, Provox® ShowerAid™, Provox® LaryTube™, Provox® LaryClip™ and Provox® LaryButton™, Provox® FlexiDerm™, Provox® OptiDerm™, and Provox® Regular™ are trademarks by Atos Medical AB, Sweden.
Provox®, Provox®2, Provox® Vega™, Provox ActiValve®, Provox® HME, Provox® HME Cap™, Provox® XtraBase®, TrachPhone®, Provox® NID™ and Provox® LaryClip™ are protected by patents, patents pending and design patents and design patents pending. Provox® XtraHME™ is protected by US patent 8,991,394, JP patent 5571687 and patents pending.
Provox® Micron HME™ is protected by US patent 8,505,537 and patents pending. Provox® FreeHands FlexiVoice™ has patents pending and design patents pending. Provox® FreeHands HME® is protected by US patent 7,370,654 and JP patent 4220157.
Remove™ and Skin-Prep™ are trademarks owned by Smith & Nephew Inc., U.S.A.
Kapi-Gel® is a registered trademark of Kapitex Healthcare Ltd.
Liberty™, TruTone™ and SolaTone™ are trademarks owned by Griffin Laboratories.
Barton-Mayo™ and Barton-Mayo Tracheostoma Button™ are trademarks owned by Medical Innovations International, Inc.
Stoma Care, Skin Tac and TacAway are manufactured by Torbot Group Inc.
Servox® is a trademark owned by Servona GmbH.
ADDvox™ and Clarus™ are trademarks owned by Bruce Medical.
Blom-Singer® is a trademark owned by InHealth Technologies.



U.S.:

Atos Medical Inc.

11390 West Theodore Trecker Way
West Allis, WI 53214-1135
Customer Service: 1-800 217 0025
info.us@atosmedical.com
www.atosmedical.us

Canada:

Ontario Home Health

29 Monteith Avenue
Stratford, ON N5A 7J9 CANADA
Tel: 1-866 964 5984 ext 2100
sales@ont-home-health.on.ca

Head office:

Atos Medical,
Hyllie Stationsväg 42,
SE-215 32 Malmö, Sweden

Manufacturer:

Atos Medical AB,
Kraftgatan 8,
P.O. Box 183,
SE-242 22 Hörby, Sweden

Tel: +46 (0)415 198 00
info@atosmedical.com
www.atosmedical.com



©Atos Medical AB, 201503, REF 8002US

Atos Medical products and expertise are developed in close cooperation with leading institutions, doctors, researchers, speech language pathologists and patients from all over the world. Our claims and arguments rest on the results of clinical studies. Atos Medical started the research and development on the first Provox voice prosthesis in 1987. Throughout the years, we have worked hard to develop Provox as the leading voice and pulmonary rehabilitation system on the market.

ATOS MEDICAL